


Orange Link Getting Started Guide

- 1.) Press the small button on the front of the OrangeLink to open the battery lid.
- 2.) Insert (2) 1.5v DC AAA Batteries into the OrangeLink, see below for (-) and (+) orientation, and close the lid.



- 3.) Enable the OrangeLink in your Loop Application:

Settings > Pump > Devices - Orange (enable toggle switch ON)

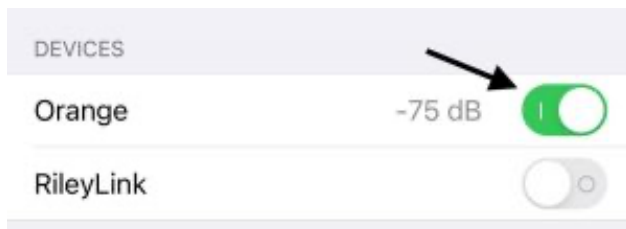

- 4.) To RESET the OrangeLink. Open the battery lid and then close it again.

LED and Haptic (Vibration) Feedback Indications

1. When OrangeLink is first powered on, LED light will blink Red and Yellow with a strong vibration.
2. When OrangeLink is powered on but not connected to a phone the LED light will blink Red every 10 seconds
3. When OrangeLink is first connected to a mobile phone, a SOFT vibration is triggered.
4. When the OrangeLink is connected to a mobile phone, LED light will blink Yellow every 10 seconds
5. When OrangeLink is disconnected from a mobile phone a STRONG vibration is triggered.
6. When battery is low, LED light will be constant RED, and with SOFT vibration every 2 mins.
7. When the OrangeLink has a "super low" battery, the voltage is too low to trigger constant RED and SOFT vibration, you will see the OrangeLink repeatedly (connect then disconnect) from the phone (See below for Super Low Indication)